

Troubleshooting

SCREENS	DESCRIPTION	IMPLICATION/ACTION
<p>Press YES to Resume No for New Program/ (Syringe)</p>	<p>Something has occurred which has interrupted the current programme (e.g. syringe displaced/power failure) so the device is prompting the user to their intention.</p>	<p>Pressing "YES": will continue current, interrupted infusion. Check/confirm infusion summary screens and press YES to restart current infusion. Pressing "NO": will programme a new infusion e.g. new syringe and or new patient. The pump will calculate the volume of the syringe and based on duration required will start a new programme.</p>
<p>Pump paused too long</p>	<p>Pump has been left in STOP mode (on hold) for 2 minutes.</p>	<p>Either: start the infusion, continue pause or turn the pump off.</p>
<p>Low battery</p>	<p>Alert: Battery is almost depleted.</p>	<p>Prepare to change battery.</p>
<p>Near end (nearly empty)</p>	<p>Alert: Infusion will end soon.</p>	<p>Prepare to change syringe or turn pump off.</p>
<p>End battery</p>	<p>Alarm: Battery is depleted.</p>	<p>Change battery.</p>
<p>End Program</p>	<p>Alarm: Infusion is complete.</p>	<p>Close down or start new infusion. (Pump will default to KVO program me if set).</p>

<p>Syringe displaced, check syringe</p>	<p>Alarm: One or more of the syringe detection sensors is not detecting.</p>	<p>Check the syringe and re-seat as necessary.</p>
<p>Occlusion/syringe empty. Check line and syringe. Press YES to confirm</p>	<p>Alarm: - Patient access device blocked - Clamp on the infusion line - Tubing occluded - Pump has reached the minimum travel position</p>	<p>- Flush or replace device/SC site - Release the clamp - Clear the occlusion - Turn pump off and manage with PRN's</p>
<p>Near System error. Press and hold INFO for details. If problem persists, send pump for service</p>	<p>Alarm: System error</p>	<p>Pressing INFO key will display the reason for the alarm and give advice for correction, if applicable.</p> <p>If correction not possible:</p> <ul style="list-style-type: none"> • Remove pump from use and turn power off • Return to technical department or CME McKinley • Include explanatory note of the error code and a brief description of the problem
<p>Keypad locked</p>	<p>To activate or deactivate the keypad lock.</p>	<p>Press and hold information button for approximately 5 seconds.</p>

TURNING OFF A S/C PUMP

When turning off a s/c pump which you wish to resume at the current regime, you **MUST** press the **STOP** button **BEFORE** pressing the on/off button.

If you do not do this, the pump must be redrawn to obtain the correct mls/hr.



